


# MANUAL FOR **MODI BUGGY**

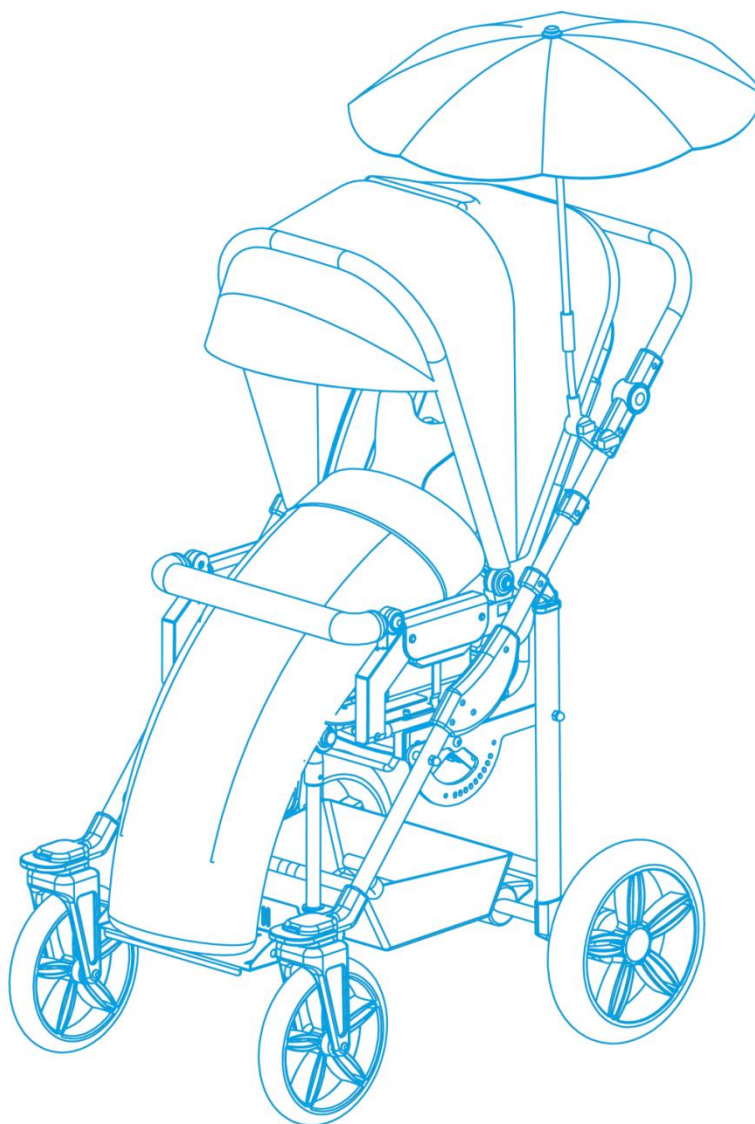

*Sizes: MINI, STANDARD*



***Edition 9 – 2021-05-26***



**NOTE! THE MANUFACTURER IS SOLELY RESPONSIBLE ONLY FOR THE EQUIPMENT PURCHASED FROM A DIRECT REPRESENTATIVE OF LIW CARE TECHNOLOGY, OR IN SPECIALIZED MEDICAL SHOPS REPRESENTING LIW CARE TECHNOLOGY IN POLAND.**

1.	Introduction .....	3
2.	General safety standards .....	4
3.	Indications for product use .....	5
4.	Weight and dimensions .....	6
5.	Structure of the Modi Buggy .....	8
6.	Detailed mounting description of Modi Buggy .....	10
7.	Moving the Modi Buggy .....	11
8.	Detailed structure and adjustment description of Modi Buggy .....	12
8.1	Unfolding the buggy's frame .....	12
8.2	Folding the buggy's frame .....	13
8.3	Parking brake .....	14
8.4	Back plate angle adjustment .....	15
8.5	Push handle adjustment .....	15
8.6	Seat angle adjustment .....	16
8.7	Seat depth and hip supports adjustment .....	17
8.8	Height adjustment of the backrest .....	18
8.9	Loins support adjustment .....	19
8.10	Footrest height adjustment .....	19
8.11	Side supports adjustment .....	20
8.12	Head supports adjustment .....	21
8.13	Adjusting and removing the roll bar .....	22
8.14	Regulation, mounting and dismantling the tray table .....	23
8.15	Mounting the rain cover .....	25
8.16	Mounting the leg cover .....	25
8.17	Mounting the umbrella .....	26
8.18	Mounting and dismantling the canopy .....	26
8.19	Footrest adjustment .....	27
8.20	Footrest supporting legs adjustment .....	27
8.21	Abduction belts installation .....	28
8.22	Vest attachment and adjustment .....	29
9.	Maintenance and cleaning .....	31
10.	Warranty/Service .....	32
11.	Identification plate .....	32
12.	Symbols meaning .....	33
13.	Compliance with the safety requirements for medical devices .....	33



Developed and created by LIW Care Technology, first-class Pediatric Pushchair Modi Buggy (hereinafter referred to as the buggy), is designed to transport children who are not heavier than 30kgs). It is designed to facilitate the daily care of children who need transport, whose diseases enable their self-sufficient movement as well as the correct sitting and standing. Buggy Modi medical device is designed for children with paralysis and triplegia or tetra- plegia; with diplegia, hemiplegia or hemiparesis. We did our best to make this Modi Buggy Pediatric Pushchair really easy to operate, to provide stability, with the ability to adapt to the body shape.

We are aware of the fact that assuming sitting position by a disabled person is a very important element of psychomotor development and the opportunity for comfortable and safe movement in open space has a huge, positive impact on patients and their caregivers alike. That is why, we have Modi Buggy, which was designed to be easy to use and to dismantle. The buggy may be used both indoors and outdoors.

We pay special attention to high quality of our products, that is why we would like you to read these user instructions thoroughly – this will ensure a safe use of this product, it will affect its efficiency and it will allow you to fully exploit the advantages of this Modi Buggy.

Keep these instructions in a safe place, so that they are available if required. These instructions are intended to be read by persons responsible for taking care of disabled children.

## 2. General safety standards



For LIW Care Technology Sp. z o. o., the most important thing is to protect the patients who use our equipment. For ensuring the safety of Modi Buggy users, the following recommendations should be adopted:

1. Please read the Instructions before the first use. In case any questions or doubts should arise, please contact your dealer or the manufacturer. Contact details are available via our website: [www.liwcare.pl](http://www.liwcare.pl)
2. Please make sure that all the information, tips and warnings contained in these chapters are fully understood.
3. Never leave your child in the buggy unattended.
4. Improper use of the buggy may endanger human health and it may cause injuries.
5. While using and operating the buggy as well as during its setting and adjusting the mechanisms, you should pay close attention to moving parts which create risk of trapping the body in holes or between the elements. After every single adjustment, you must stabilize its position by carefully tightening screws and ensure that all the components are correctly positioned and locked.
6. If the child is in a buggy, make sure it is properly secured by belts and a vest.
7. The Modi Buggy can be used by one person at a time.
8. It is forbidden to transport the baby in the Modi Buggy while using the public and private transport, ie. bus, tram, trolley bus, train.
9. It is forbidden to transport the baby in the Modi Buggy while driving a car – the seat of the buggy is not a car seat. The child cannot stay in the seat while driving.
10. In the backseat there is a pocket with all the tools that are needed to carry out the adjustments: allen wrench no. 4, flat wrench (10 mm).
11. It is forbidden to drive up and drive the buggy down stairs with both the child and without the child.

In these instructions for LIW Care Technology Sp. z o. o. products, you will find information marked with „NOTE!“ symbol. It is used to draw reader's particular attention to the text. It looks as follows:



**NOTE! THE SYMBOL IS USED TO DRAW THE READER'S ATTENTION TO THE TEXT WHICH IS MARKED WITH THIS SYMBOL. FAILURE TO COMPLY WITH THE INSTRUCTIONS MAY ENDANGER THE HEALTH AND SAFETY OF THE USER OR IT MAY DAMAGE THE PRODUCT.**



### 3. Indications for product use



Modi Buggy medical device is designed for children suffering paralysis and triplegia or tetraplegia; from diplegia, hemiplegia or hemiparesis.

It is recommended for children who have been diagnosed with:

- cerebral palsy (paralysis cerebri infantum),
- muscular dystrophies,
- paralysis,
- spina bifida,
- myelomeningocele,
- spinal injury,
- traumatic brain injury,
- strokes,
- other diseases with paresis, paralysis, locomotory disabilities enabling their self-sufficient movement.

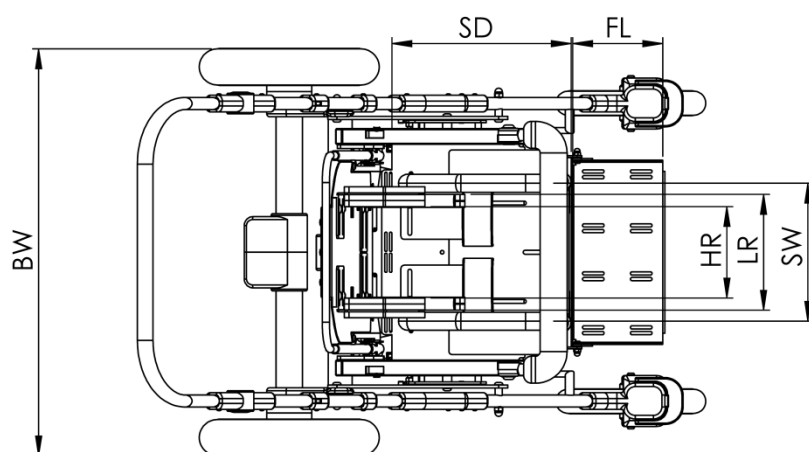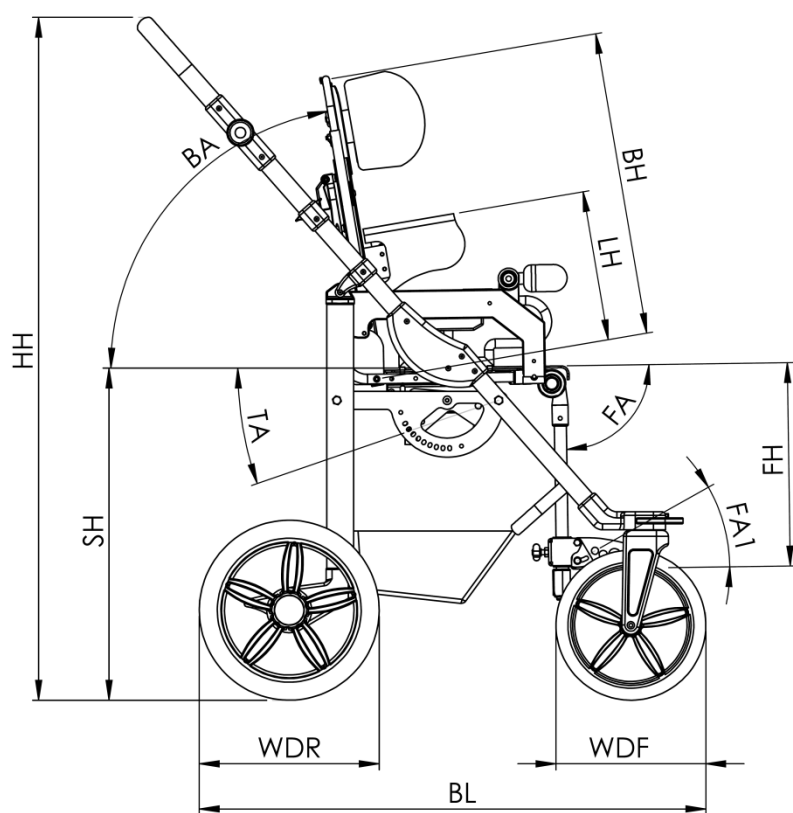

No.	Dimension	Dimension symbol	Dimension [cm]	
			Size: Mini	Size: Standard
1	Pelvic support spacing (seat width)	<b>SW</b>	15÷28	17÷30
2	Seat depth	<b>SD</b>	17÷28	27÷36
3	Turning angle	<b>TA</b>	-45°÷15°	-45°÷15°
4	Back height	<b>BH</b>	44,5÷60	50÷65
5	Back-support angle	<b>BA</b>	20°÷90°	20°÷90°
6	Side supports length	<b>LH</b>	14÷26	16÷30
7	Side supports spacing	<b>LR</b>	15÷27	15÷27
8	Head supports spacing	<b>HR</b>	15÷20	15÷20
9	Footrests height	<b>FH</b>	17÷36	17÷36
10	Footrest length	<b>FL</b>	15	15
11	Footrest angle adjustment	<b>FA</b>	0°÷90°	0°÷90°
12	Footrest assembly angle	<b>FA1</b>	0°÷20°	0°÷20°
13	Seat height	<b>SH</b>	59	59
14	Base length	<b>BL</b>	84	84
15	Base width	<b>BW</b>	68	68
16	Wheel diameter (front)	<b>WDF</b>	24	24
17	Wheel diameter (rear)	<b>WDR</b>	30	30
18	Handle height	<b>HH</b>	85÷117	85÷117
19	Maximum user's weight		30 kg	30 kg
20	Maximum load (user's weight + additional load)		33kg	33 kg
21	Weight of the buggy		20kg	20 kg





**Fig. 1** Structure of the buggy

**The wheelchair consists of<sup>1</sup>:**

1. Push handle
2. Back plate
3. Seat
4. Head supports
5. Side supports
6. Hip supports
7. Roll bar
8. Frame
9. Front wheels, Rear wheels
10. Parking brake
11. Rear shock absorbers

<sup>1</sup> The equipment of wheelchair is available depending on the Continent, Country.

- 12. Footrest
- 13. Vest
- 14. Pelvic belts
- 15. Basket
- 16. Canopy
- 17. Leg cover
- 18. Rain cover
- 19. Bag
- 20. Umbrella
- 21. Table
- 22. Loins support

## 6. Detailed mounting description of Modi Buggy



For safety reasons, your buggy is supplied in two parts: the frame of the buggy (8), and the seat (3). To install the seat, you should:

**STEP 1.** Unfold the frame (8) (→ See 8.1, p. 12)

**STEP 2.** Attach the seat (3) to the frame (8) so that the mounting bars are in the seat holders, *Fig. 2*. You need to push it until it snaps.



**Fig. 2** *Seat attachment*



**NOTE!** YOU NEED TO WORK WITH THE SEAT UNTIL YOU HEAR A DEFINITE SNAP—THE SOUND PROVES THE ATTACHMENT WAS DONE PROPERLY.




**Fig. 3** *The structure of the buggy allows you to attach the seat either facing or back to the direction of travel*

## 7. Moving the Modi Buggy



For safety reasons, the buggy must be moved with its parts, separately the seat and separately the frame.

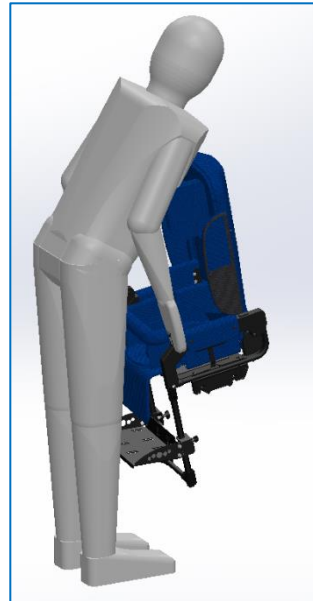

**Fig. 4** Correct manner of moving the buggy



**Fig. 5** Incorrect manner of moving the buggy



**NOTE!** AFTER EVERY SINGLE ADJUSTMENT, YOU NEED TO MAKE SURE THAT ALL THE COMPONENTS ARE CORRECTLY POSITIONED AND LOCKED.


### 8.1 Unfolding the buggy's frame



**Fig. 6** Unfolding the buggy's frame

**Fig. 7** Lock

You need to set the frame vertically *Fig. 6*, *Fig. 7* unlock it and pull the handle forcefully, until you hear a snap.

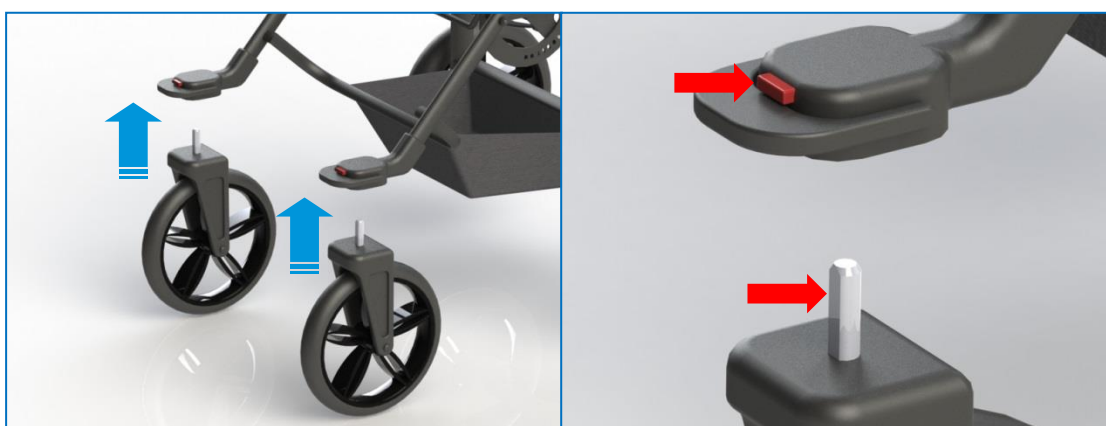

**Fig. 8** Mounting front wheels

**Fig. 9** Front Wheel bolts

Lift the front part of the buggy, *Fig. 8*, and install the front wheels: make sure that the flat surfaces of the bolts are directed towards the front of the buggy, *Fig. 9*. To remove the wheel, press the red bolt release button.



**Fig. 10** Direction lock

You can see direction locks on the front wheels in the straight-ahead position. To use them, you need to pull the lever and turn it down. By lifting the lever up, you can easily change direction, *Fig. 10*.

## 8.2 Folding the buggy's frame




**Fig. 11** Wheel dismounting

Raise the buggy and remove the front wheels, by pressing the red button at each wheel – *Fig. 11*.



**Fig. 12** Removal and installation of rear wheels

Easy and safe removal and installation of wheels is possible due to application of a special mechanism. To release a wheel press a pin located in a wheel hub with your thumb inwards and remove the entire wheel from the axis. Follow the same steps with installation – press a pin and place the wheel on the axis, *Fig 12*.

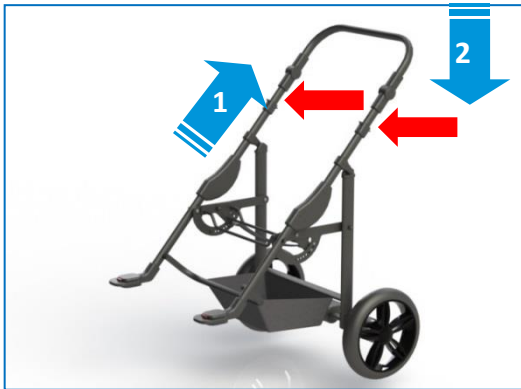

**Fig. 13** Folding the frame

Pull the handholds on both sides, and then push the handle down and fold the frame until the lock engages, *Fig. 13*.

### 8.3 Parking brake



**Fig. 14** Parking brake

The rear wheels lock is activated with a lever on the cross-bar, right on the back of the frame. By pressing it, you lock the rear wheels, by lifting it up, you release the break, *Fig. 14*.



**NOTE!** YOU SHOULD INSPECT THE BRAKE AT REGULAR INTERVALS AND REMOVE POTENTIAL DIRT AND MUD.



**NOTE!** YOU SHOULD BE CAREFUL CROSSING THE THRESHOLD.



**NOTE!** STABILITY OF THE STROLLER MAY BE AT RISK IN THE EVENT OF UNEXPECTED PUSH, TILTING OR LEANING ON IT.


#### 8.4 Back plate angle adjustment



**Fig. 15** Back plate

Lift the lever on the back of the seat and move it to set the appropriate seat-back angle up to  $20^{\circ}$ ÷ $90^{\circ}$  Fig. 15.

#### 8.5 Push handle adjustment

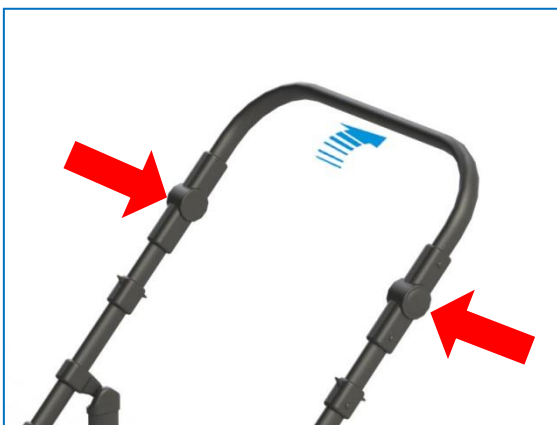

**Fig. 16** Push handle


Press the height control buttons simultaneously, set the bar to the desired height. Next, move it up and down until button lock clicks (Fig. 16). You can adjust the height of the handle up to 85÷117 cm above ground.



**NOTE!** PLEASE NOTE THAT EVERY SINGLE THING YOU PUT ON THE HANDLE AFFECTS THE STABILITY OF THE BUGGY.



## 8.6 Seat angle adjustment

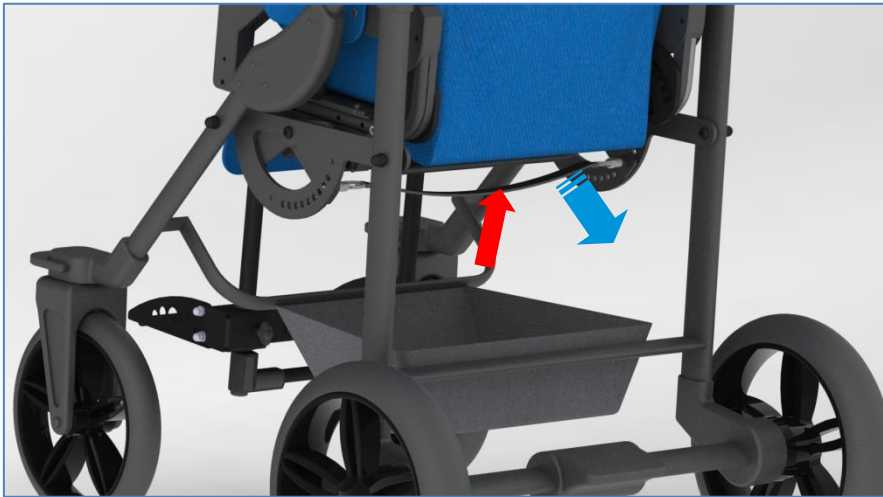

**Fig. 17** Seat angle release




**Fig. 18** Seat angle adjustment

Hold the seat, pull the control line for releasing the locking device, *Fig 17*, while positioning the seat angle from  $15^{\circ}$  in the driving direction to  $45^{\circ}$  in the opposite direction *Fig. 18*.



**NOTE!** DUE TO A VERY HIGH DEGREE OF ADJUSTABILITY OF THE BUGGY, THERE IS A POSSIBILITY TO SET A POSITION, WHERE THE HEAD IS PLACED BELOW THE PELVIS. IT IS FORBIDDEN TO USE THAT POSITION, BECAUSE IT MAY LEAD TO REGURGITATION OR CHOKING.

## 8.7 Seat depth and hip supports adjustment

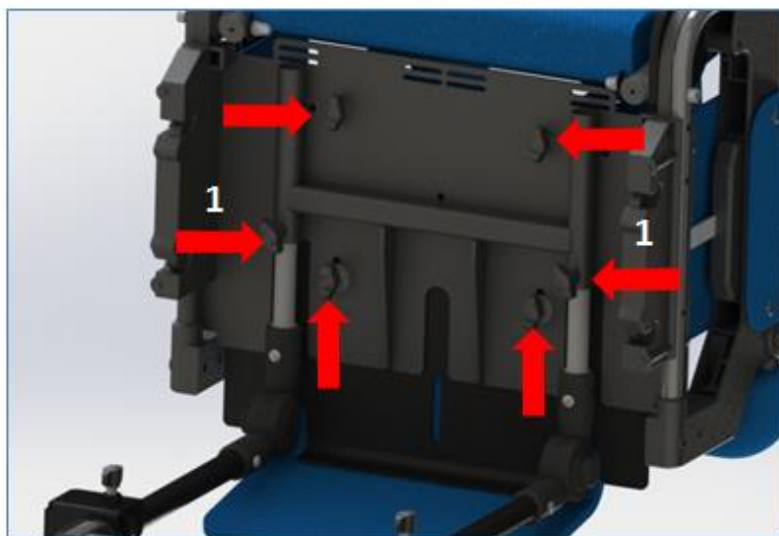

**Fig. 19** Seat locking knobs

Loosen all locking knobs *Fig. 19*.



**Fig. 20** Seat depth adjustment


Pull the seat base to a desired position, *Fig. 20*, and then tighten the knob (1) until there is noticeable resistance. *Fig. 19*.



**Fig. 21** Hip supports adjustment

Before re-locking the other knobs *Fig. 19*, slide apart/ slide hip supports. Their design allows an inclination to ensure the right tight pulling angle *Fig. 21*.


### 8.8 Height adjustment of the backrest

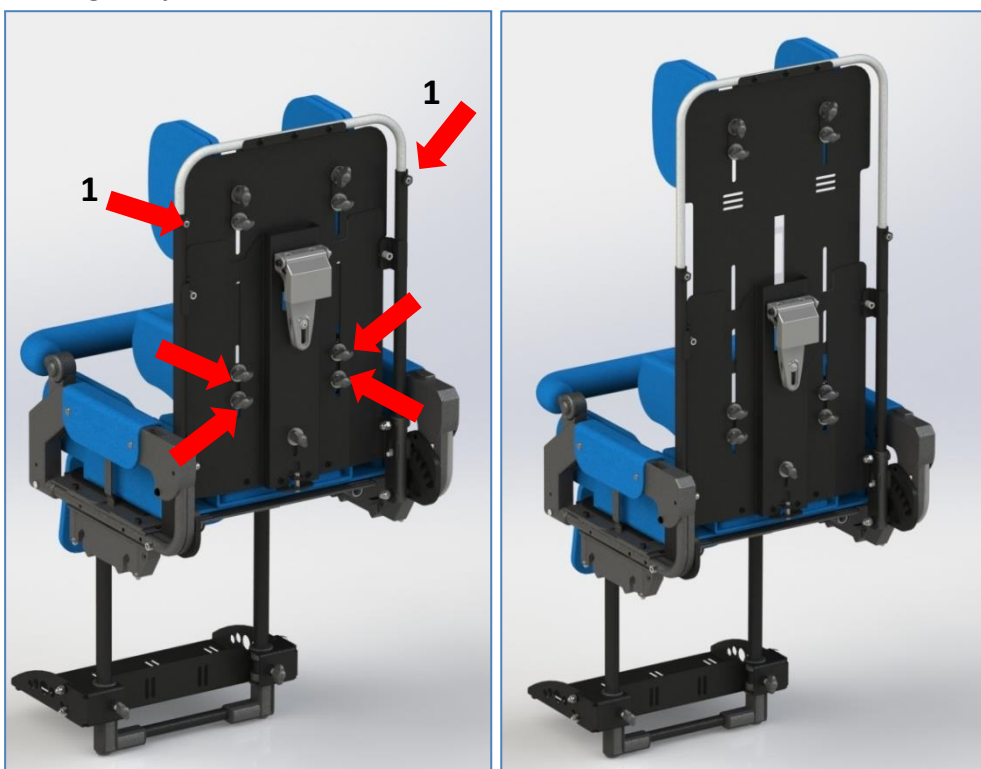

**Fig. 22** Backrest height adjustment

Unscrew the fixing screws (1) and the locking knobs *Fig. 22* to set the desired height of the backrest in the range of 50÷65 cm. Then tighten the screws (1) and locking knobs.

## 8.9 Loins support adjustment

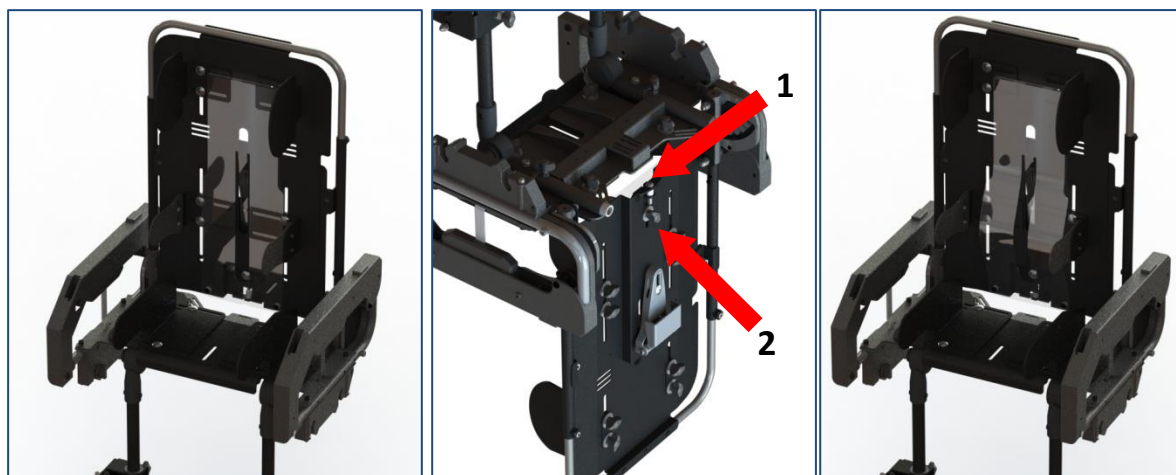

**Fig. 23** Loins support adjustment

In Modi Buggy, there's an option of cambering the back support. To adjust loins support, you should loosen the knob **2** (fig. 23) and next – using the allen key 4mm – tighten the bolt **1**, until we obtain back support required for the patient. After adjustment, tighten the knob **2**.

## 8.10 Footrest height adjustment




**Fig. 24** Footrest height adjustment

Loosen the locking knobs *Fig. 24*, set the desired height of the footrest and tighten the locking knobs. It is possible to set footrest height in the range: 17÷36 cm.



**NOTE!** THE FOOTREST ONLY SUPPORTS THE FEET, YOUR CHILD SHOULD NOT STAND ON IT.

### 8.11 Side supports adjustment

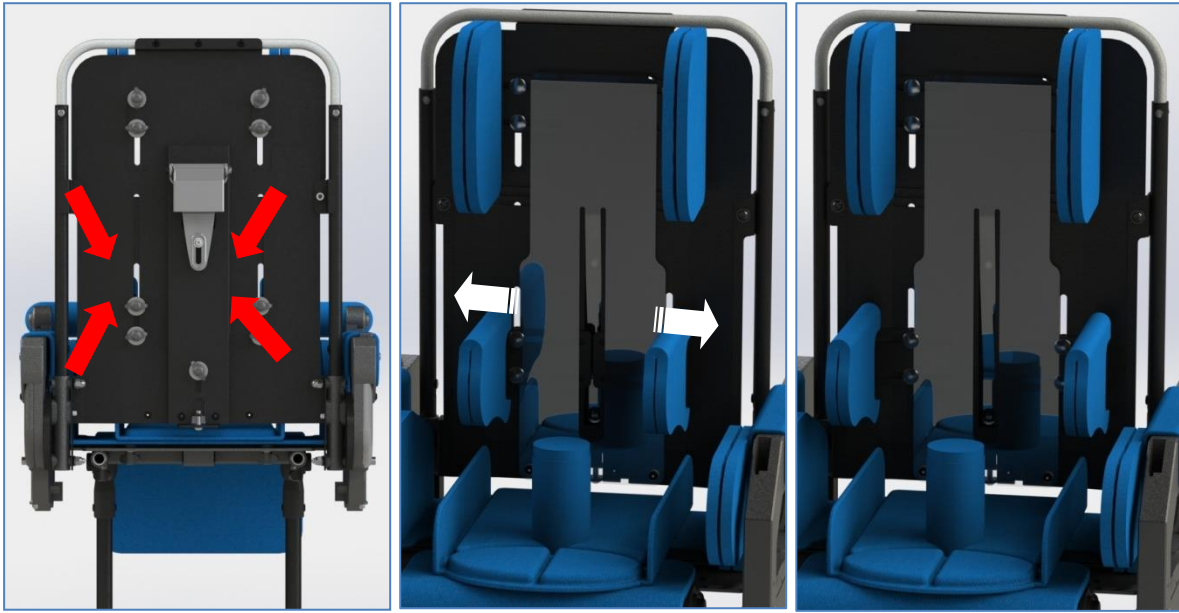

**Fig. 25** Width adjustment


Loosen the 4 knobs *Fig. 25*, move the supports up to: 15÷27 cm.



**Fig. 26** Height adjustment

Pelletes height adjustment possible in the range of 16÷30 cm for standard size and 14÷26 cm for the mini size (*Fig. 26*).



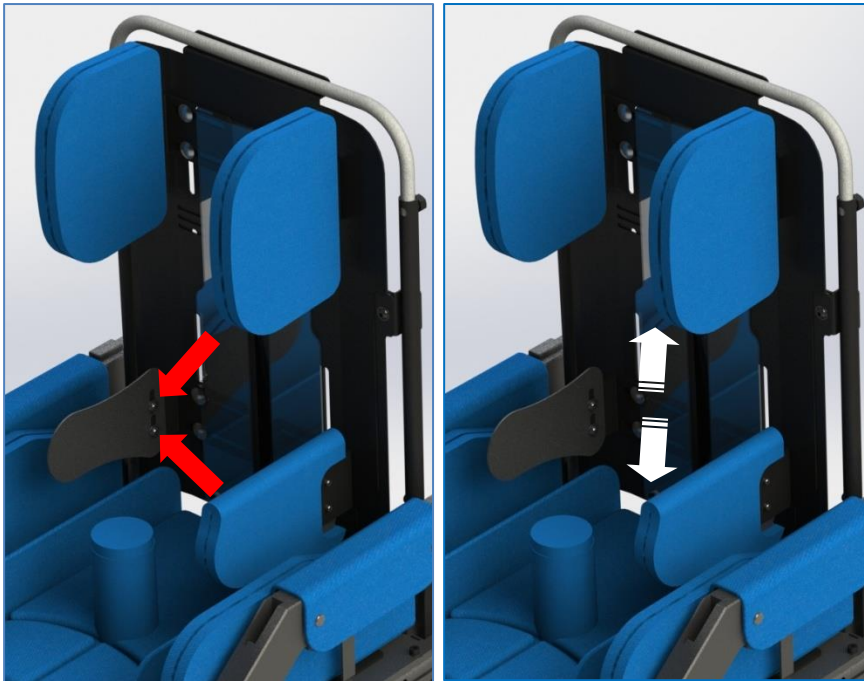

**Fig. 27** Asymmetrical height adjustment

For asymmetrical height adjustment of your side supports you need to loosen the locking screws and set the desired height of the supports. *Fig. 27*. Next tighten the locking screws. Maximum height difference is 20 mm. Greater asymmetry can be achieved by changing the side supports' height loosening the locking knob *Fig. 26*.

#### 8.12 Head supports adjustment




**Fig. 28** Adjusting the width of the head supports

Loosen the locking knobs *Fig. 28* and set the desired width of the head supports in the range of 15 to 20 cm and its height  $\pm 2$  cm *Fig. 28*. Then, tighten the locking knobs.




**Fig. 29** Head supports height adjustment

### 8.13 Adjusting and removing the roll bar

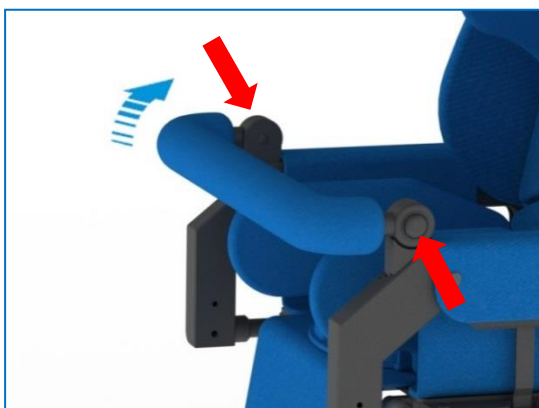

**Fig. 30** Roll bar adjustment


Press the buttons on both sides simultaneously and set the required roll bar position, *Fig.30*.

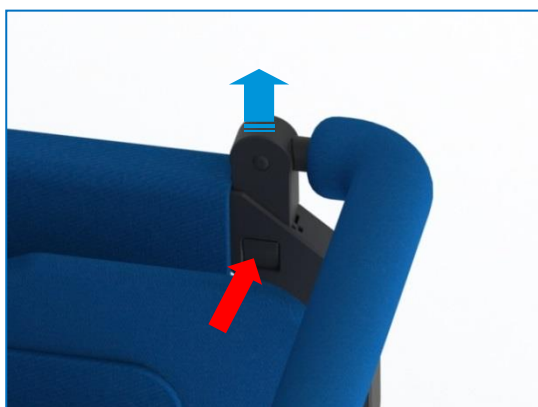

**Fig. 31** Removing the roll bar

Press the clip, firmly grasp the bar and pull it out, *Fig. 31*.

#### 8.14 Regulation, mounting and dismantling the tray table

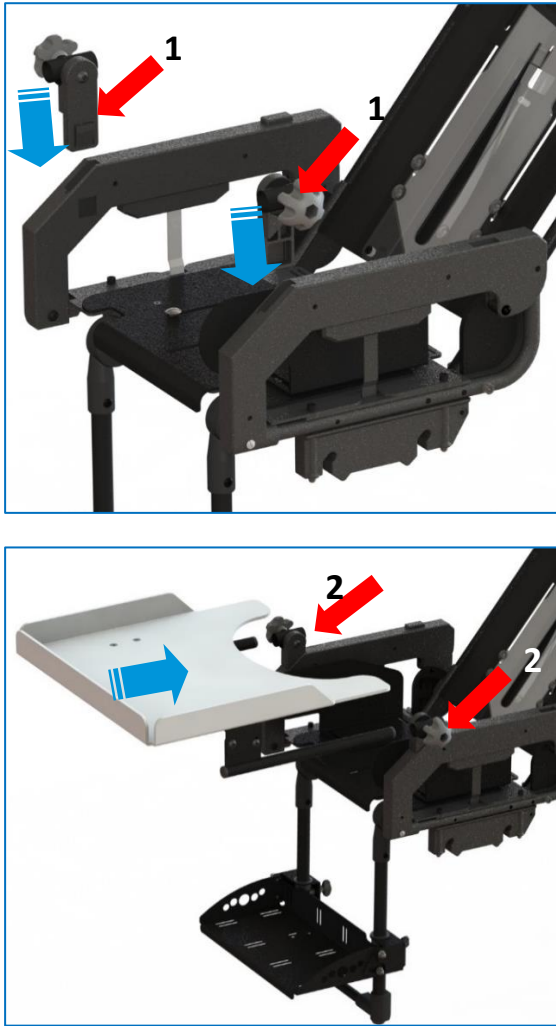

**Fig. 32** Mounting the tray table



To mount the table, insert the table fastening adapters 1, then slide in the table using the tubes into the adapters. Tighten the knobs 2, to prevent the table from slipping out.



**Fig. 33** Table Angular Adjustment



Angular adjustment of the worktop is made by loosening the two screws located above the table holders below the worktop. After obtaining the desired angle of inclination, tighten the screws. The adjustment screws are located on the left and right side of the table.

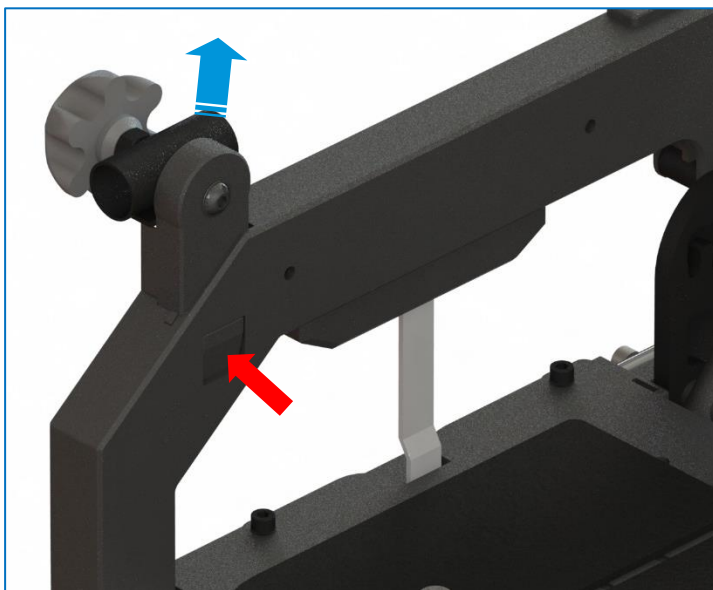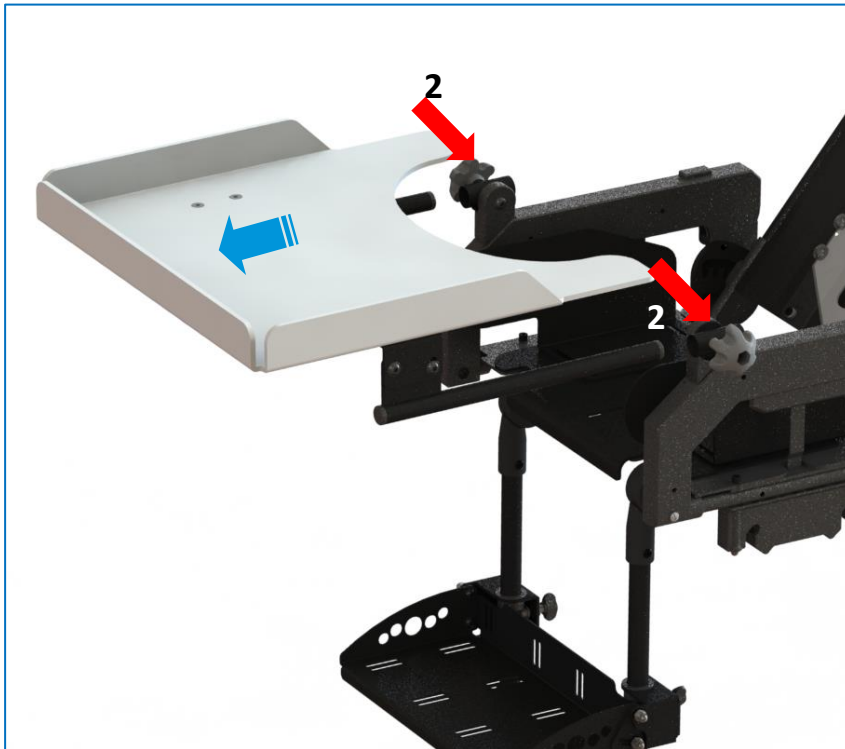

**Fig. 34** Dismantling the table

To remove the table, unscrew the knob (2) Fig. 34, then pull out the table. To remove the adapter, press the button (3), then pull up the adapters on both sides of the seat.


### 8.15 Mounting the rain cover



**Fig. 35** Mounting the rain cover

Fasten the rain covers using the elastic edge by placing it behind the back support (1) and the footrest (2).

### 8.16 Mounting the leg cover

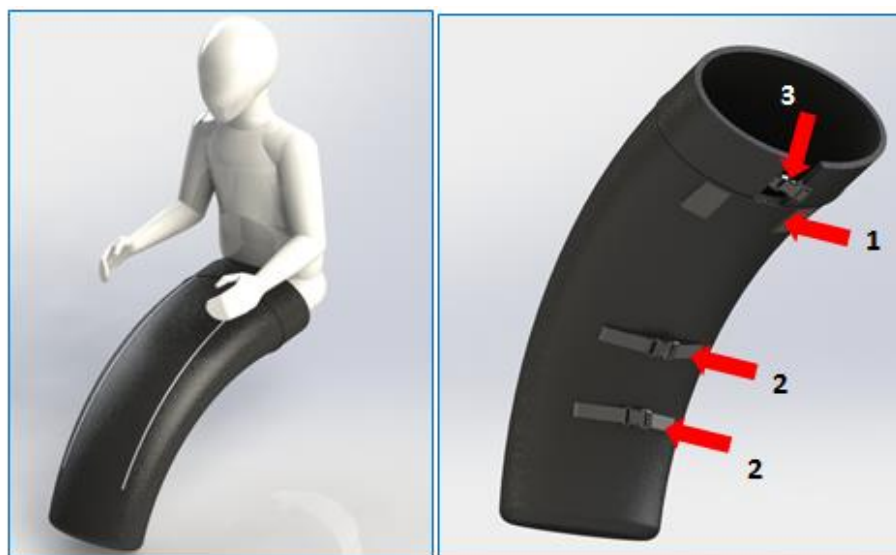

**Fig. 36** Mounting the leg cover

Push the leg cover under the seat upholstery and fasten using the Velcro (1), fasten the buckle (2) covering the push chair's footrest. After placing the baby in the stroller, fasten the buckle (3) circumscribing the strap behind the back rest.

### 8.17 Mounting the umbrella



**Fig. 37** Mounting the Umbrella

Umbrella should be fixed with the clamp (1). Place the clamp on the buggy's frame and tighten the knob (2). To change the angle of the umbrella, unscrew the knob (3), adjust the angle of the umbrella, and then tighten the knob again (3).

### 8.18 Mounting and dismantling the canopy



**Fig. 38** Mounting and dismantling the canopy

Canopy latches (1) should be inserted into the socket boxes on both sides of the seat. In order to dismantle canopy, press the latch lock (2) on both sides of the seat, and then pull out the canopy upwards.


### 8.19 Footrest adjustment



**Fig. 39** Footrest angle adjustment

Loosen the bottom screws, set the right angle and re-lock the screws. *Fig. 39*. It is possible to adjust the footrest angle in the range:  $0^{\circ} \div 20^{\circ}$ .


### 8.20 Footrest supporting legs adjustment

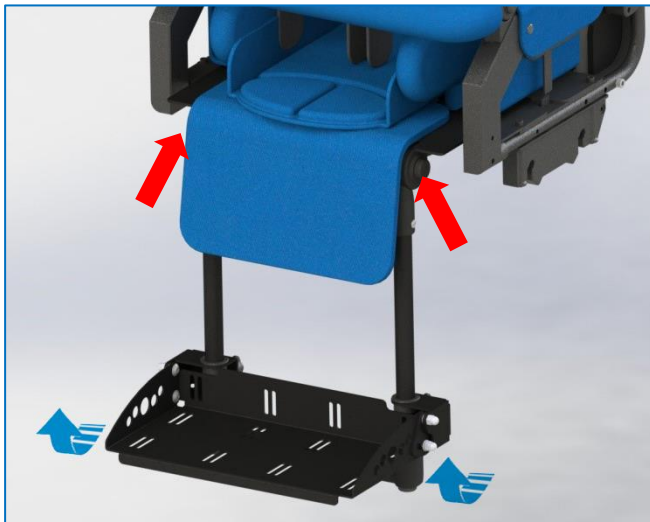


**Fig. 40** Footrest supporting legs angle adjustment

Press the two release buttons at the same time *Fig. 40* then adjust the angle of the footrest brackets in the range:  $0^{\circ} \div 90^{\circ}$ .

## 8.21 Abduction belts installation

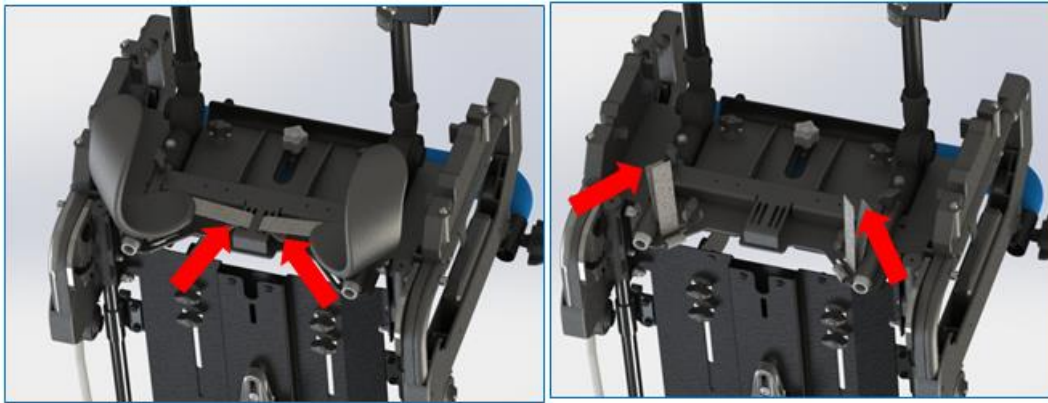



**Fig. 41** Abduction belts installation for the mini size

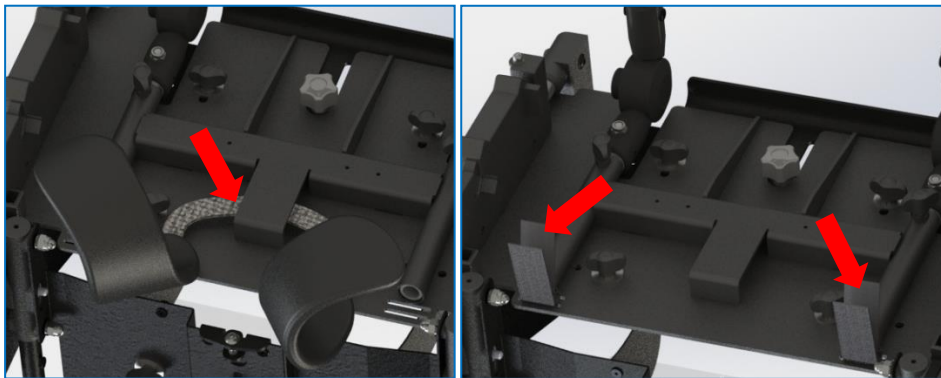

**Fig. 42** Abduction belts installation for the standard size


Depending on seat size, seat belts come in two options. The version with cuts comes with the mini size, in which one end of the abduction belt should be installed in the slots from the Fig. 41 on the right. For the standard size abduction belts are stitched together, and because of that they should be putted underneath the element of seats construction (Fig 42). Another end of the abduction belts should be installed inside the slots under the seat. The excess of the belts is used to install the vest.

## 8.22 Vest attachment and adjustment

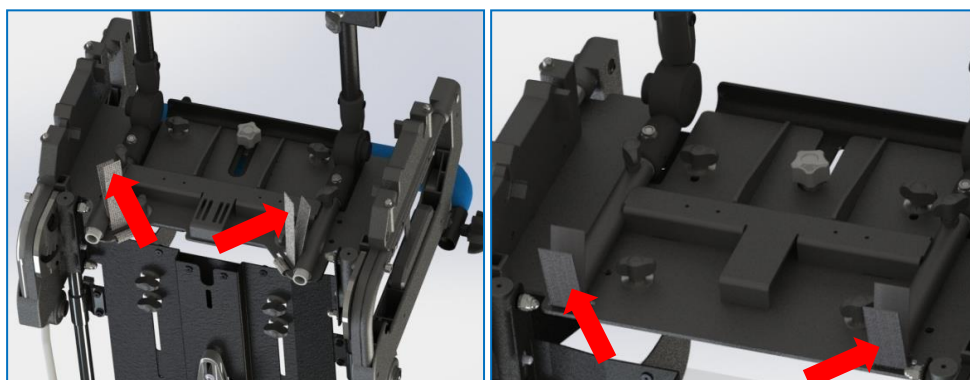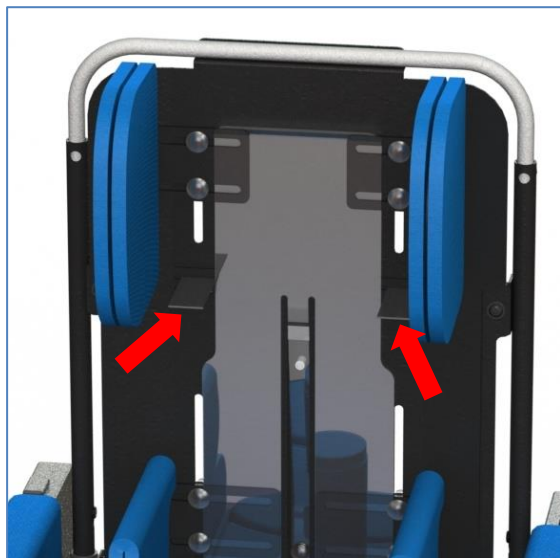

**Rys. 43** Fixing the straps of the vest


The upper straps attaching the vest should be placed on chosen construction element of the backrest (Fig. 43) at the level close to the one of the child's shoulders. Bottom straps of the vest should be putted in the slots under the seat (the belt is common for vest and abduction belts). Next they should be taken through the slots in the seats upholstery. Clampers should be installed on the edges of the straps.

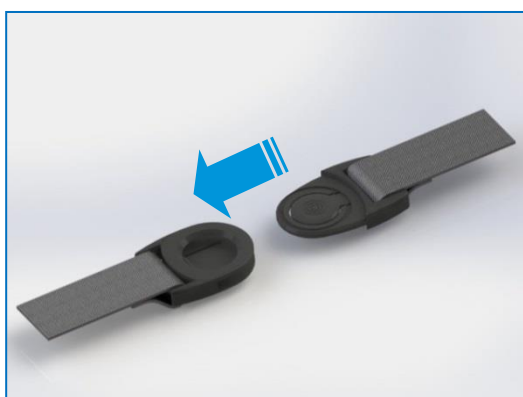

**Fig. 44** Clips fastening



The vest should be fastened after the child has taken the seat, slide the clips until you hear a click. *Fig 44.*

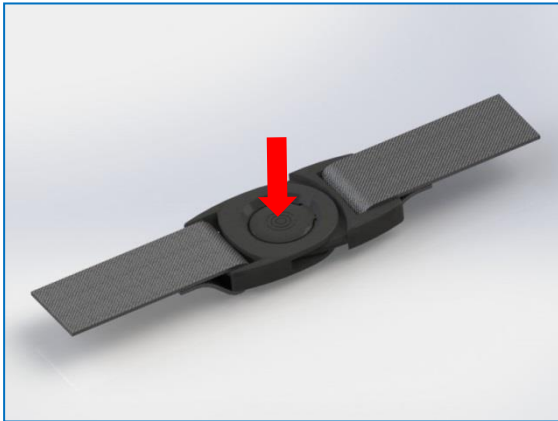

**Fig. 45** *Clips unfastening*

To unfasten the vest, press it right in the middle to unlock the clip. *Fig. 45.*

You control the vest by adjusting the length of the fastening straps. Proper adjustment of the vest should provide the right stability of the child's body, but it should not cause any discomfort.



**NOTE! ALWAYS MAKE SURE YOU FASTEN THE CLIPS PROPERLY, USING ALL FOUR CLIPS, BECAUSE OTHERWISE, A LACK OF FASTENING MAY CAUSE THE CHILD TO FALL OUT OF THE BUGGY.**



Modi Buggy is a mechanical device with a supporting structure made of steel and aluminum, covered with a varnish coat. Foam-sponge inserts are connected to the structure. Foams are covered with a dust cover made from textile materials. None of the coatings cause skin irritation. Modi Buggy, like any other medical device, should be kept in necessary purity and used in accordance with the manufacturer's instructions.

- Varnish coats and dust covers made from textile materials, should be cleaned with a cloth dampened with water. It is allowed to use mild detergents or enzymatic cleaning agents. Clean this if necessary with a vacuum cleaner or a soft brush.
- It is allowed to clean the foam-sponge inserts with a cloth dampened with water and with mild detergent. After carrying out this action, the insert shall be thoroughly dried at room temperature.



**NOTE! THE MODI BUGGY UPHOLSTERY SHOULD BE COMPLETELY DRY BEFORE BEING USED AGAIN.**

- You should protect the frame regularly, remove dirt and mud regularly.
- Do not use chemicals.
- Do not wash the upholstery in a washing machine.
- You should avoid avoiding direct sunlight and check the seat temperature before putting the baby inside.



**NOTE! MODI BUGGY SHOULD BE SERVICED AT LEAST ONCE A YEAR (EVERY 12 MONTHS) BY A QUALIFIED SERVICE TECHNICIAN. IT IS NECESSARY TO CARRY OUT PERIODIC INSPECTION OF THE BUGGY – MOVING PARTS, LOCKING MECHANISM, ADJUSTMENT FUNCTIONS. PERIODIC REVIEW OF THE DEVICE ENSURES A LONG AND TROUBLE-FREE OPERATION.**



## 10. Warranty/Service



Should you notice any faults or defects, you should stop using the buggy immediately and contact your dealer or manufacturer. Defective unit must be protected against enlarging the area of damage. Never attempt to disassemble or repair the product. Do not replace original parts with the ones coming from a source other than the manufacturer recommends.

If the user decides not to continue using the product it is bound to its disposal in accordance with the applicable environmental regulations.

The economic lifetime of the product is five years.

The manufacturer provides post – warranty service. Contact details:

LIW Care Technology Sp. z o.o., ul. Golfowa 7, 94-406 Łódź, Poland.

biuro@liwcare.pl

Current contact details are available on: [www.liwcare.pl](http://www.liwcare.pl)

Warranty terms are specified in the warranty card, which is an integral part of this statement. The warranty card is available on the last page of this document.

## 11. Identification plate



MODI BUGGY		CE
SN	MB - 10000	
rozmiar/size:	STANDARD	
	30 kg	
MD		
LIW Care Technology Sp. z o.o., ul. Golfowa 7, 94-406 Łódź, Poland		

MODI BUGGY		CE
SN	MB - 20000	
rozmiar/size:	MINI	
	30 kg	
MD		
LIW Care Technology Sp. z o.o., ul. Golfowa 7, 94-406 Łódź, Poland		

## 12. Symbols meaning



Manufacturer's name



Date of manufacture



Serial Number



User's permissible weight



Note! Follow product instructions



Avoid contact with water



Unlocking direction



Movement direction



Mark of conformity according to the Regulation 2017/745 of the European Parliament and of the Council (EU) dated from April 5<sup>th</sup>, 2017 on medical devices, Annex V.



Medical device

## 13. Compliance with the safety requirements for medical devices



Hereby we confirm that the Modi Buggy meets requirements of the Regulation of the European Parliament and of the Council (EU) 2017/745 dated from April 5<sup>th</sup>, 2017 on medical devices. The Modi Buggy in accordance with Annex VIII of the Regulation of the European Parliament and of the Council (EU) 2017/745 dated from April 5<sup>th</sup>, 2017 on medical devices is a non-invasive, active class I medical device according to the rule 1.

The Declaration of Conformity for this product can be found in manufacturer's Sales Department.

## WARRANTY CARD

### Warranty terms:

1. Manufacturer of medical device - LIW Care Technology Sp. z o.o. (ul. Golfowa 7, 94-406 Łódź, Poland) ensures that the company sells an operative device, free from construction, installation, material defects, etc., and that the device will remain free from such defects during the warranty period. This warranty does not cover usefulness of the product for the Buyer purposes.
2. LIW Care Technology Sp. z o.o. (the manufacturer) grants a two-year warranty for the equipment, starting from the initial purchase date.
3. The only proof of warranty is this warranty card, issued by the Seller to the Buyer. To transfer warranty entitlements, you need to transfer the possession of this warranty card.
4. In case of finding any faults or defects during the warranty period - the manufacturer will remedy the defect at his own cost. If parts are replaced, the removed part shall become the property of the LIW Care Technology Sp. z o.o. and shall not be returned to the warranty user.
5. Under the warranty, the user is obliged to report physical product defect during the warranty period via a form on our website: [www.liwcare.pl](http://www.liwcare.pl), following prior logging to the "customer zone" by post LIW Care Technology Sp. z o.o. or by phone: +48 42 212-35-18
6. Under the warranty, the user is obliged to deliver the equipment with warranty card and along with the original document of purchase (VAT invoice or receipt) at the expense of the manufacturer to his seat. when there is no receipt, the initial purchase date (the start of the warranty period) shall be the date of manufacture.
7. This warranty does not cover:
  - waste elements and destroyed or damaged parts due to improper use (in particular, but not exclusively due to improper instructions issued or under unfavorable conditions), or improper storage of the product,
  - damage caused by any alterations or additions to the device made by the user or a third parties,
  - damage caused by improper cleaning or maintenance of the product made by the user or a third party,
  - damage caused by normal wear and tear or normal aging of the product,
  - damage caused by user's lack of negligence (particularly, but not exclusively, in maintenance and cleaning of the product),
  - damage caused by force majeure,
  - damage caused by external events (pollutions, mechanical and water damage).
8. The warranty does not include does not include regulation and adjustment during the warranty period, because they do not constitute a product defect.
9. In order to qualify for the above warranty, you need to deliver the product which is securely enclosed in packaging to the manufacturer. To do that, use the original packaging. If the user do not deliver the product in the original packaging, his replacement packaging must satisfy the following requirements: solid box with intact walls, separate protection for each component, cushioning material, strong tape used for wrapping, as well as sender address or receiver (LIW Care Technology Sp. z o.o.) address.
- LIW Care Technology Sp. z o.o. shall not be held liable for product damage caused by insufficient packaging.
10. Warranty Repair or replacement will be made as far as possible within 30 days from the proper delivery of the product by the warranty user to LIW Care Technology Sp. z o.o.
11. After warranty repair, the product will be delivered at the expense of LIW Care Technology Sp. z o.o., to the supplied address. In the event when a correctly addressed package is not collected, the warranty user is obliged to shall bear all costs related to the product transport and its storage.
12. Quality warranty granted by LIW Care Technology Sp. z o.o. does not affect any legal rights of the Buyer.

*Thank you for choosing our product. We wish you success in therapy with our equipment.*

### NOTE!

#### KEEP THE WARRANTY CARD IN A SAFE PLACE

LIW Care Technology Sp. z o.o. will require providing this document before accepting the warranty repair.

Product name/Type .....Serial number:.....

Sold-by date:.....

Stamp and signature of the Seller: .....

No.	Date of repair request	Procedures	Repair completion date	Stamp and signature of the technician
1				
2				
3				
4				