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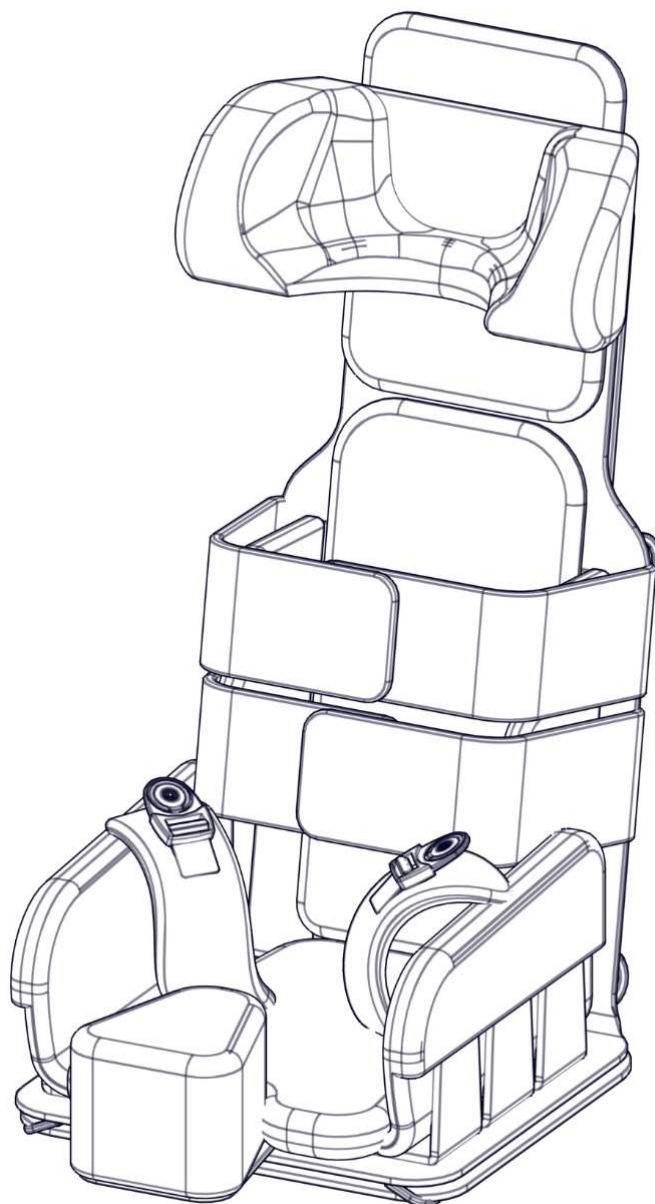
NOTE! THE MANUFACTURER IS SOLELY RESPONSIBLE FOR THE EQUIPMENT PURCHASED DIRECTLY FROM THE LOCAL REPRESENTATIVE OF LIW CARE TECHNOLOGY OR IN A SPECIALISED MEDICAL SHOP REPRESENTING LIW CARE TECHNOLOGY IN POLAND.

NOTE! IF POSSIBLE, THE PACKAGING OF THE PRODUCT SHOULD BE RETAINED IN CASE WHEN IT WOULD PROVE NECESSARY TO SHIP THE PRODUCT WITHIN THE SCOPE OF WARRANTY REPAIR.

NOTE! DO NOT LEAVE A CHILD ON THE SEAT UNSUPERVISED.

NOTE! IN OCCUR OF ANY MEDICAL INCIDENT RELATING TO THE PRODUCT, NOTIFY THE MANUFACTURER IMMEDIATELY IN WRITING BY REGISTERED POST WITH ACKNOWLEDGEMENT OF RECEIPT OR BY E-MAIL

NOTE! IT IS NECESSARY TO CAREFULLY READ THE USER MANUAL BEFORE USING THE PRODUCT, AND RETAIN IT UNTIL THE END OF ITS USE.

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1 Introduction

Thank you for purchasing the **TravelSit** offered by LIW Care Technology Sp. z o.o., an orthopaedic seat for children stabilising the back and the head. **TravelSit** is an innovative device comprising the features of an orthopaedic chair, which can be easily adjusted to meet the anatomical conditions of the user. Mobility stands as its most unique feature. This is a fully legitimate orthopaedic seat with a system of stabilising supports, abducting and stabilising straps, which can be folded into the size of a backpack. This particular feature allows to take it with you whenever you need a comfortable seat for your child. The system of stabilising straps enables you to change a standard chair or armchair into a safe orthopaedic seat.

We are fully aware of the fact that travelling not only means pleasure, but also the necessity to select all the essential items you need to take with you on the trip. All means of transportation are characterised by a limited luggage space. We have created the TravelSit so that you no longer have to face the dilemma of choosing between the safety and the comfort of your child. The TravelSit will always be there when you need it, enabling your child to enjoy the comfort

2 Warnings & Safety Information

It is essential that you read this manual before using the appliance. Follow all the instructions in the manual, you will avoid situations where you could damage the appliance and ensure your total safety and comfort during the entire life of the product.

For LIW Care Technology Sp. z o.o., the most important thing is to protect the patients who use our equipment. In order to provide complete safety of people using the device it is essential to strictly follow the recommendations stated below:

1. Before undertaking any attempts related with using the device, please read the user manual thoroughly and in case of any doubts, do not hesitate to contact the seller or the manufacturer.
2. Please make sure that all the information, tips and warnings included in these chapter are fully comprehensible.
3. The child must not be left in device without supervision by a career.
4. TravelSit is intended for use by one person at a time.
5. It is forbidden to move the patient in the device.

All modifications of the device made by customer not considered by user manual are not allowed by manufacturer and might cause warranty loss.

Pay attention to the device condition. The occurrence of any damage or signs of excessive wear on the unit

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NOTE! THIS PARTICULAR SYMBOL IS USED TO STRONGLY EMPHASISE THE FOCUS OF THE READER ON THE WORDING MARKED WITH THIS SYMBOL. FAILURE TO COMPLY WITH THE INSTRUCTIONS MAY ENDANGER THE HEALTH AND SAFETY OF THE USER OR IT MAY DAMAGE THE PRODUCT.

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should be checked and, if necessary, such an incident reported to the distributor or manufacturer.

Any incidents that occur in connection with the use of appliance will be reported to the producer of the device and to responsible authorities depending on the location where the incident occurred.

User manual attached to devices manufactured by LIW Care Technology Sp. z o.o. holds a paragraph marked with a word NOTE, which aims to emphasise the content of the given paragraph. The significance of the symbol, of which mention has been made above is as follows:

3 Indications for product use

The TravelSit can be used for people with abnormal posture and muscular dysfunction.

It stands as a perfect solution for

- children cerebral palsy,
- muscular dystrophy,
- various types of paralyses,
- tetra- and paraplegias,
- as well as for children with body posture abnormalities.

The system of stabilising supports, abducting and stabilising straps can be used as a therapeutic and prophylactic agent, as well as to prevent the occurrence and development of already existing posture defects, and the improper functioning of the organism associated with the above.

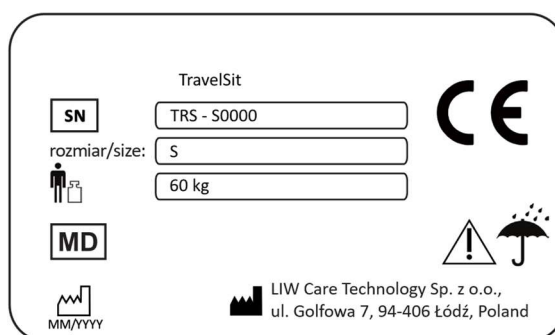
The main objective of the device is to provide passive correction of the abnormal posture the child in sitting position. The TravelSit stands as the perfect substitution of a traditional orthopaedic chair, one that you can always take with you wherever you go and whenever you need it.

The patient/user should not use the device when:

- the attending physician, in the patient's positioning recommendations, forbids the body to be placed in the positions indicated in the normal use of the product.
- medical conditions do not permit use.
- use of the device could cause the child's condition to deteriorate

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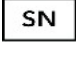
6 Compliance with the safety requirements for medical devices

Hereby we confirm that the **TravelSit** orthopaedic seat for children meets requirements of the Regulation of the European Parliament and of the Council (EU) 2017/745 dated from April 5th, 2017 on medical devices.

The **TravelSit** orthopaedic seat for children in accordance with Annex VIII of the Regulation of the European Parliament and of the Council (EU) 2017/745 dated from April 5th, 2017 on medical devices is a non-invasive, active class I medical device according to the rule 1.

Conformity declaration of the device can be obtained in the Commercial Department of the manufacturer.

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NOTE! In case of introducing any modifications to the device, using non-original spare parts or utilizing with products provided by another manufacturer, the CE marking has to be removed.

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NOTE! The TravelSit fixing system enables to install this seat on a car seat, however, this particular seat is not a car seat anticipated for carrying children in cars. This seat can only be used during standstill. The manufacturer shall not be held responsible for using this seat as a car seat.

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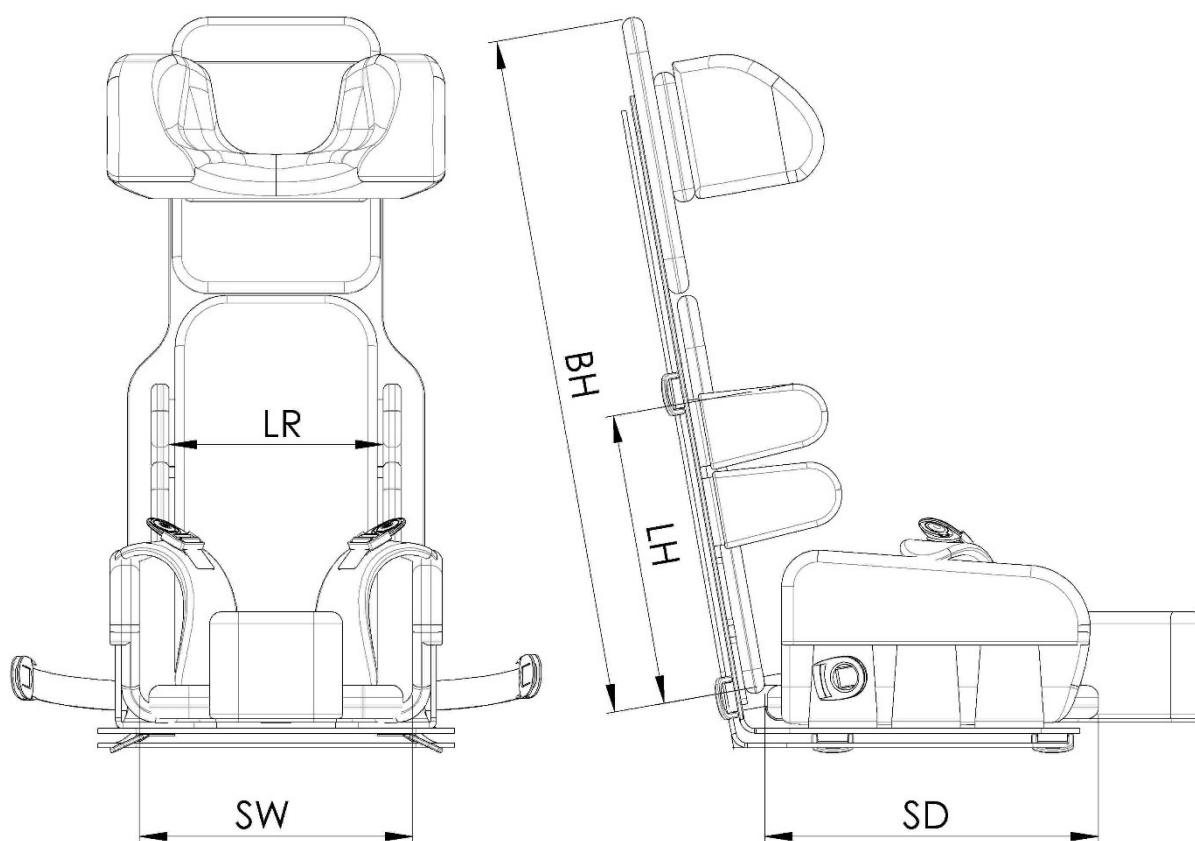
7 Application

The TravelSit orthopaedic seat for children stabilising the back and the head is designed for people, who are between 85 and 110 cm tall and whose weight does not exceed 60 kg.

The device allows to adapt a normal chair or an armchair and change it into a fully legitimate orthopaedic seat. Due to its special features you may take this seat whenever you go on a journey or visit your friends.

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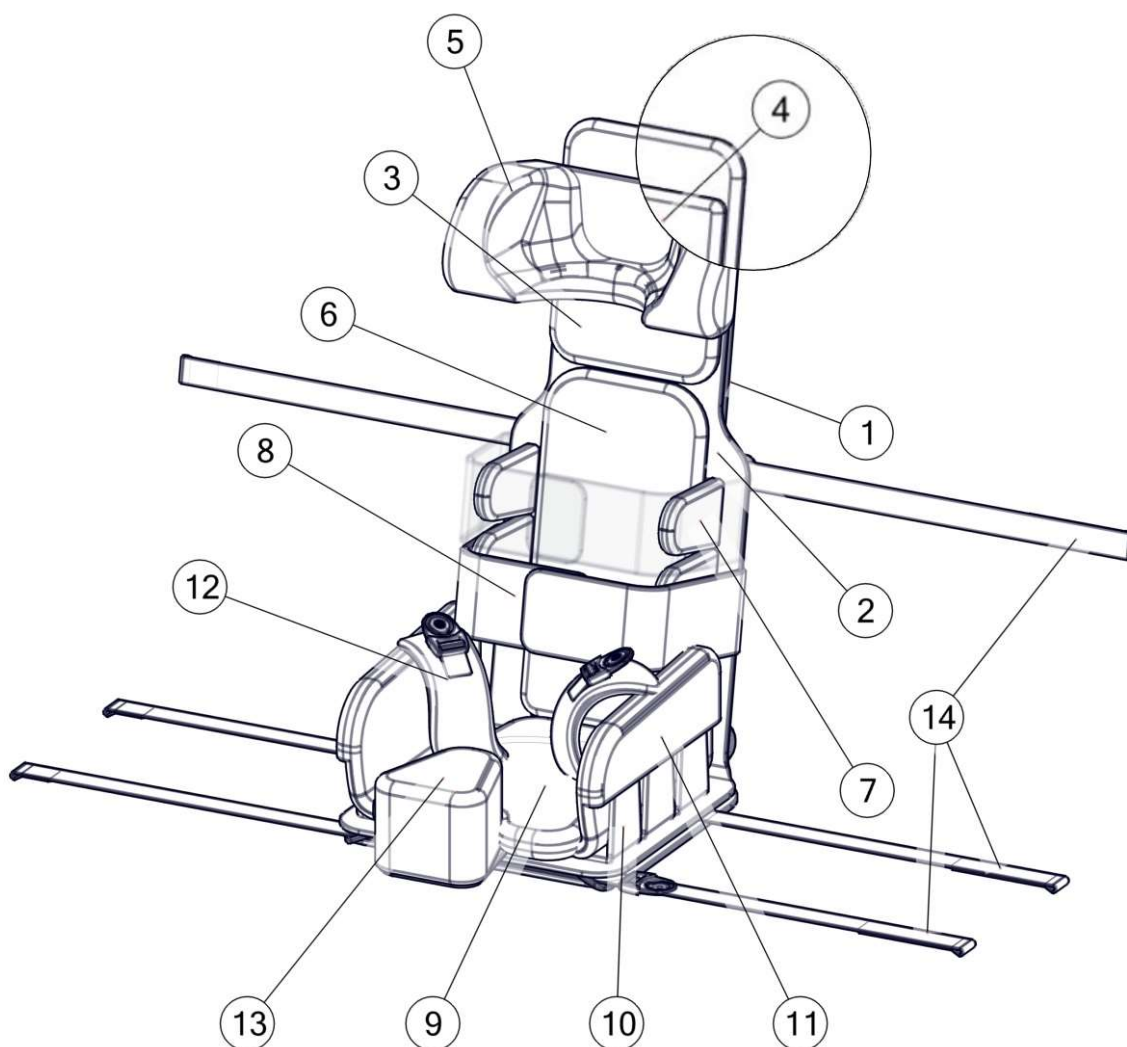
1. Mat with a seat support
2. Base mat
3. Headrest distance
4. Additional headrest distance
5. Headrest
6. Backrest cushion
7. Lateral supports
8. Chest straps
9. Seat cushion
10. Hip supports
11. Side cushion of the seat
12. Pelvic straps
13. Abductor
14. Fixing straps

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10 Detailed description concerning the regulation and the adjustment

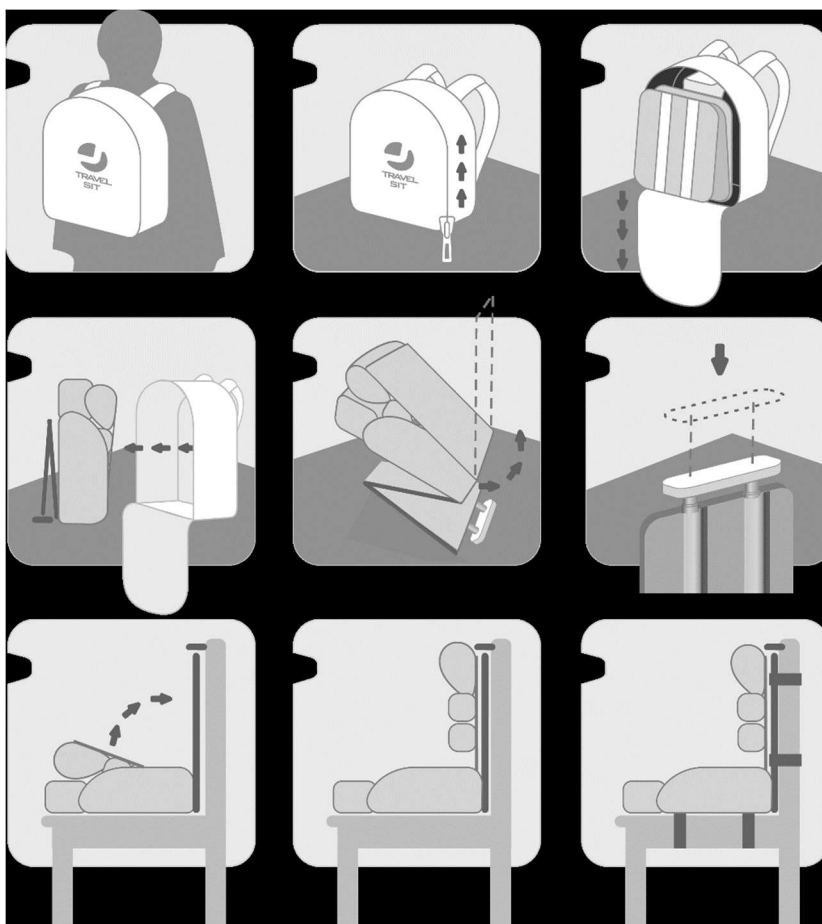
TravelSit is comprised of a system of belts, supports, cushions, brackets, etc. connected with various velcro-tightening sets. This allows to apply multiple configurations to the device and adjust the device to the anatomical structure of the user by altering the position of the system's elements on various heights and widths of the base mat. The system allows an arbitrary positioning of these elements, along with adding or deducting them. What is more, it also allows to position the fixing straps in such a way that they match various chairs, armchairs, trolleys or bases dedicated for this particular system. The manual presented below provides sample examples for adjusting and setting up the device.

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10.1 Preparing for using the device

In order to prepare the device, follow the sequence of activities shown on the pictures below.

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Whenever you want to prepare the device for transportation, perform all these activities in the reverse order.

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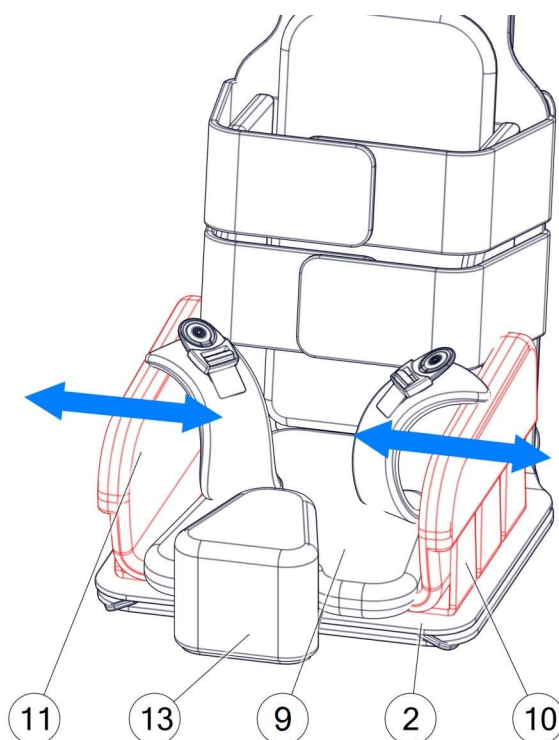
NOTE! Before using the device it is crucial to fasten the fixing belts to the adapted armchair or chair. After fastening these belts, it is essential to verify the stability of the seat on the adapted object, as well as the stability of the whole construction.

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10.2 Adjusting the width of the seat

In order to adjust the width, detach the cushions of the seat (9), abduction wedge (13) and hip support (10) from the base mat (2). Place thigh supports (10) to reach the desired width. Thigh supports (10) have velcro pads at the bottom. Then reposition the abduction wedge (13) and the seat cushions (9).

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
10.3 Adjusting the position of side support

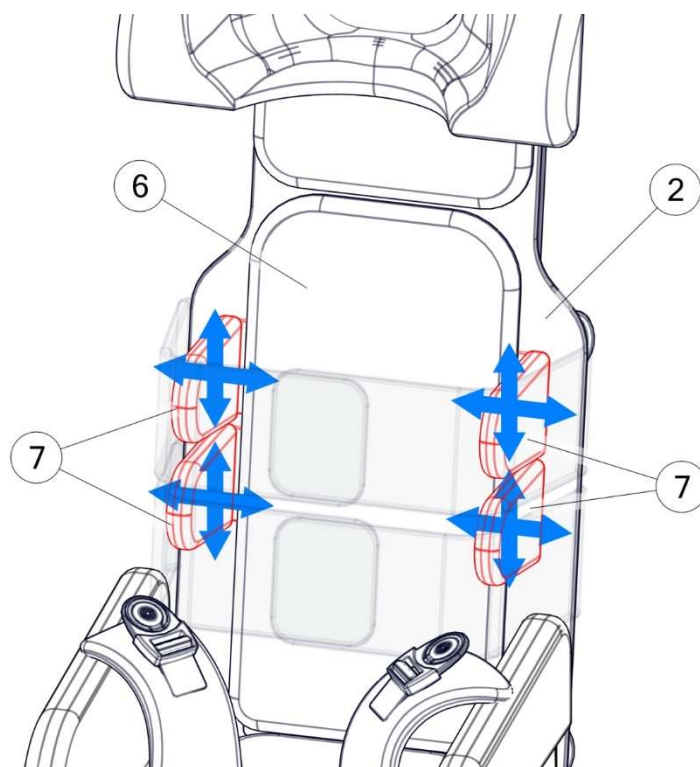In order to adjust the position of the side support detach the cushions (6) of the backrest along with lateral supports (7). Move lateral supports (7) to the desired location and push them firmly to the base mat (2). Lateral supports (7) have velcro pads in their lower parts.

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
10.4 Adjusting the position of the abduction wedge

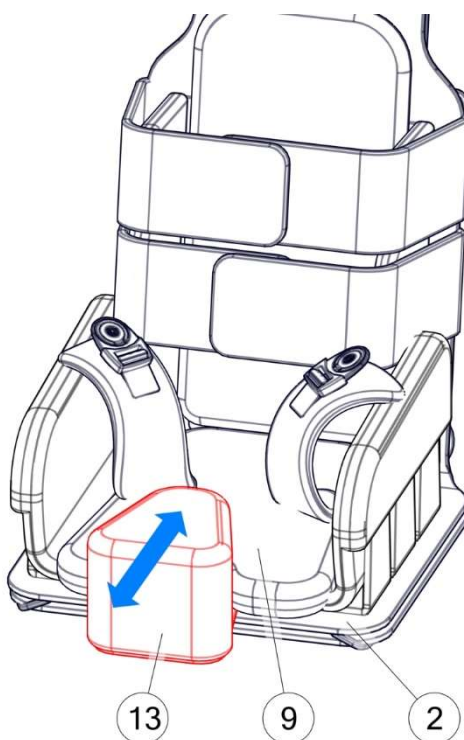In order to shift the abduction wedge, detach the seat cushions (9) and then shift the wedge (13) inwards or move it outwards and then press to the base mat (2) after reaching the desired position.

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
10.5 Adjusting the chest straps and fixing straps.

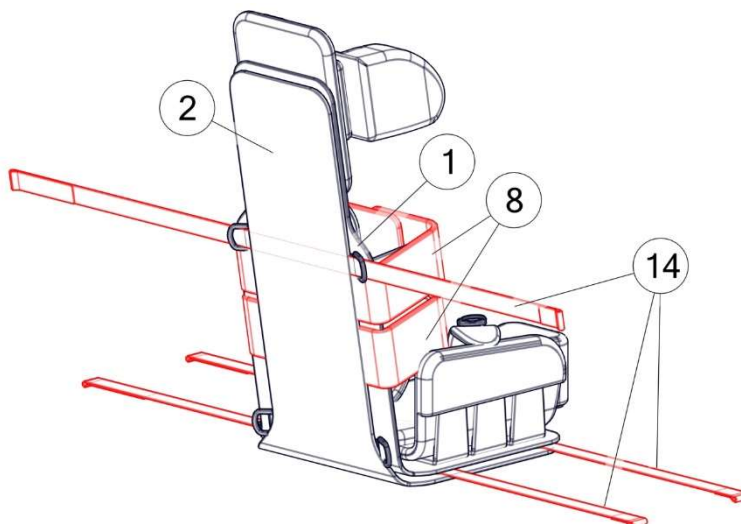In order to shift the chest harness straps (8) detach the base mat (2) along with the mat with a seat support (1). Detach the chest harness straps (8) and move the mat to a desired position. Reattach the base mat (2) and the mat with a seat support (1). Do the same with the fixing straps (14).

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
10.6 Adjusting the position of the headrest

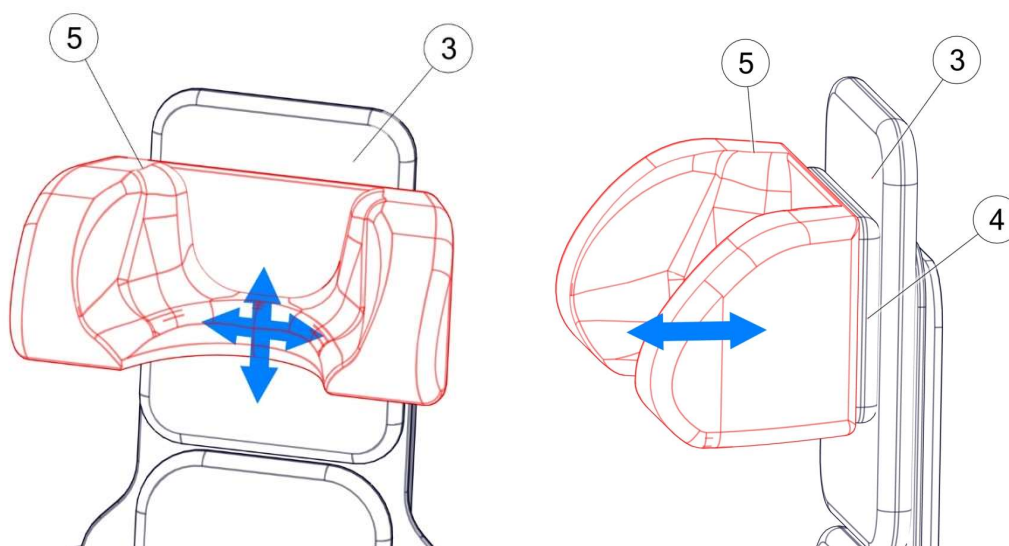In order to change the position of the headrest (5), detach the headrest from the seat (3), move it upwards or downwards, and then attach the headrest in the desired position with a velcro tape. You can also change the position of the headrest moving it forward or backward by adding or deducting additional headrest cushions (4).

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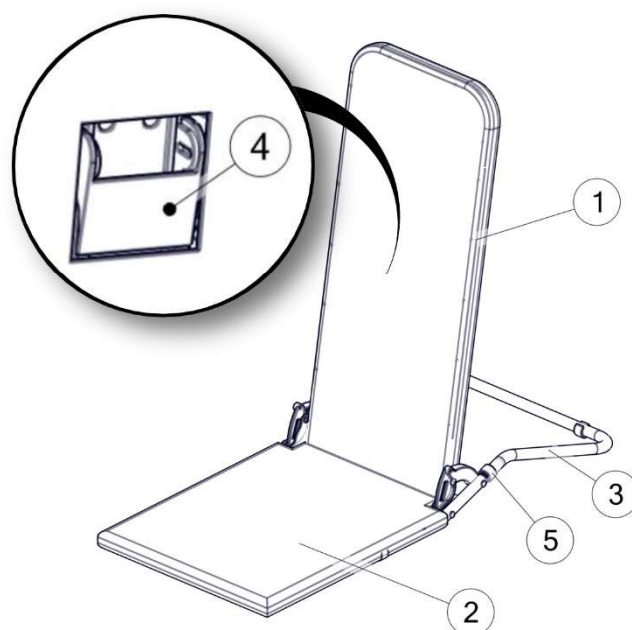
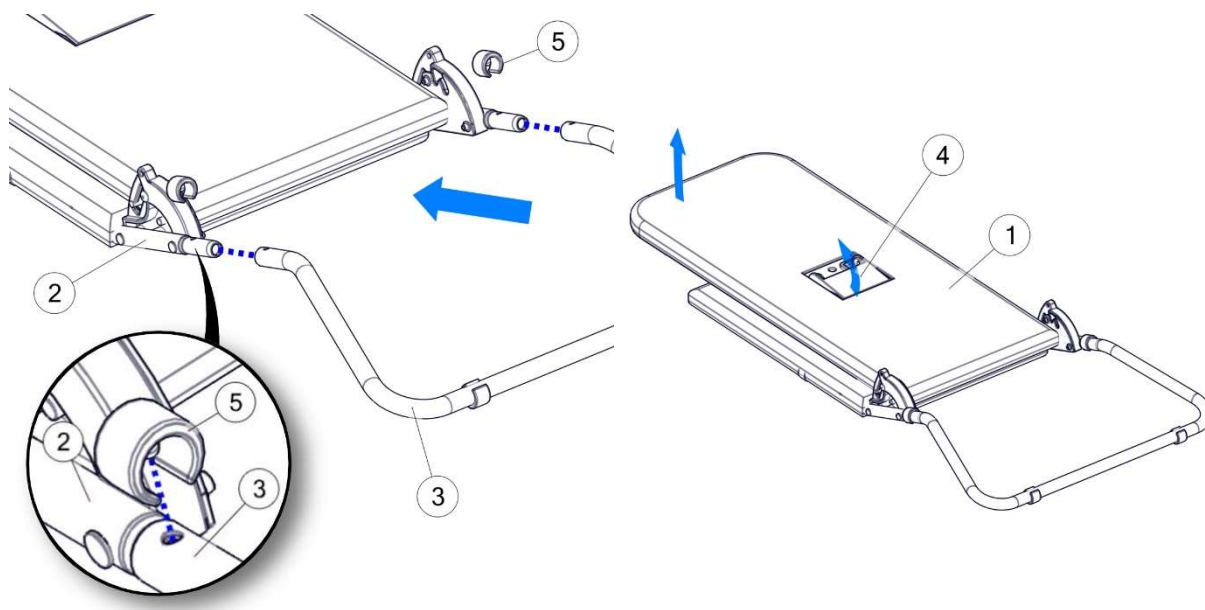
11.1.2 Preparing the lounger

In order to prepare the lounger for use, it is essential to insert the bracket (3) into the frame of the seat (2) and secure it with tabs (5). Then we pull the retarder (4) and lift the backrest (1) until it reaches a vertical position. Then detach the TravelSit seat from the support mat, and connect the TravelSit with the lounger by means of sewn-in velcro pads.

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
11.1.3 Folding the lounger

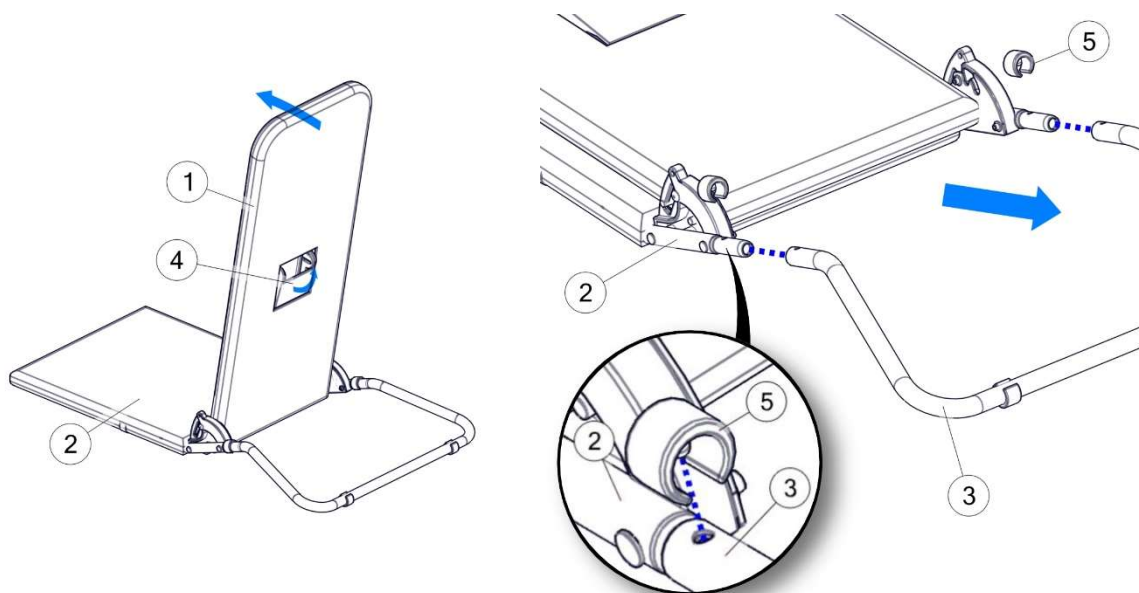We pull the backrest angle regulation retarder (4), we fold the backrest towards the seat (2). Then we insert tabs (5) and we disassemble the bracket.

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11.1.4 Adjusting the seat back angle

In order to regulate the backrest angle, pull the retarder (4), adjust the backrest angle (1). When the lever (4) is released the backrest shall be blocked.

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NOTE! Before using the lounger, please verify the stability of the lounger.

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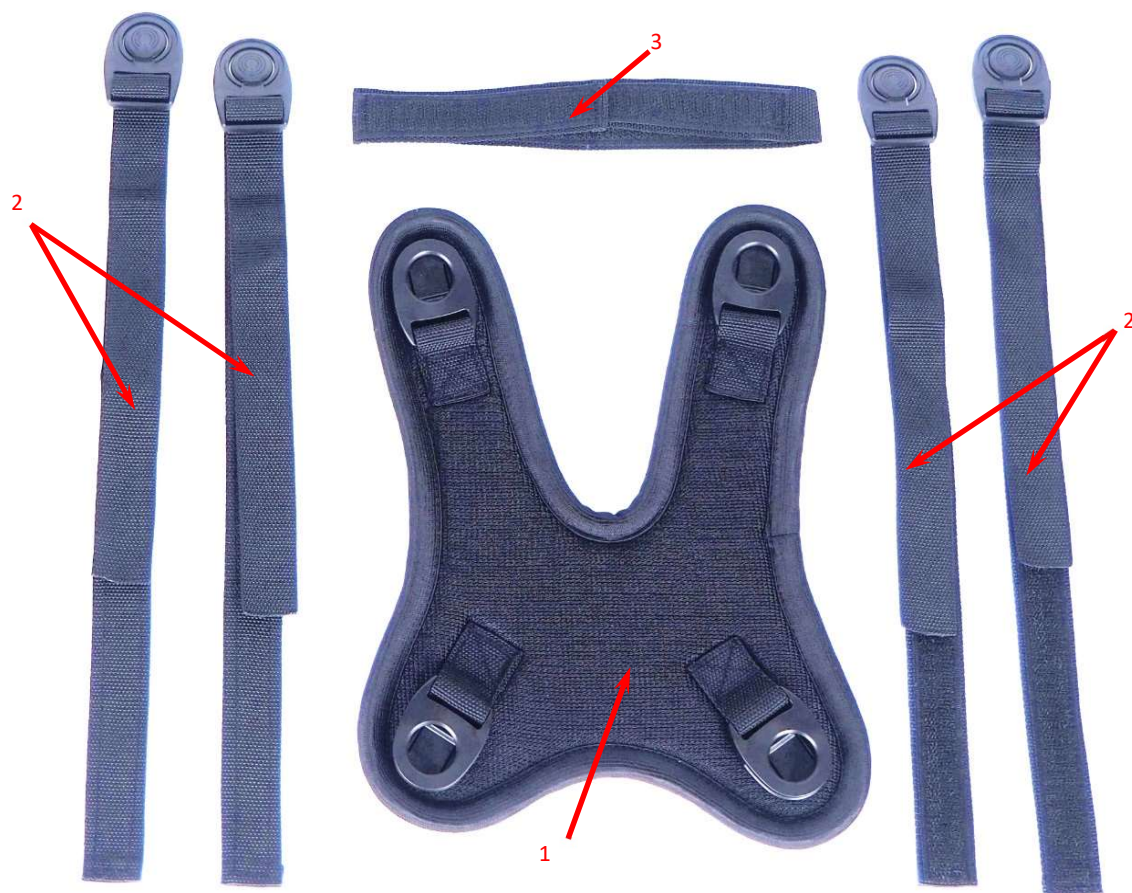
11.2 Positioning vest

The vest is used to provide the proper position of the child.

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11.2.2 Vest's assembly

Detach the back cushion (2) off the base mat (1). Next, mount the adjusting straps (3) to the base mat (2) by pressing them. Rys. 14

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The reinforcing strap (1) hitch to the base mat (2) which will immobilise the adjusting straps (3). The ends of the adjustable straps with claspers should be left on the top of the reinforcing strap. Loose ends of the reinforcing strap should be mounted on the back of the seat. Rys. 15

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Side supports (1) should be adjusted in required position and mounted to the base mat (2) by pressing them. Next the back cushion (4) should be added. The ends of the straps (3) stays on the top, together with the vest's mounting claspers. Rys. 16

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To mount the bottom adjustment straps (3) the bottom part of the back cushion (1) should be taken off together with the Velcro strap (2) from the base mat (4). Next, the adjusting straps (3) should be mounted by pressing them to the base mat (4) on the both sides of the seat. Rys. 17

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Stabilising of the placement for the bottom adjustment straps (1) should be done by pressing them to the base mat (2) with the use of Velcro strap (3). Next, the back cushion (4) should be released down, and pressed to the Velcro strap. Rys. 18

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The positioning vest (1) should be mounted to the clamps of adjusting straps (2) as presented on Rys. 19

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12 Spare parts and consumables

The manufacturer does not provide for the replacement of components by the end customer. Repairs and replacement of components should be carried out by technically competent persons so as not to damage the equipment or pose a risk to life or health during replacement.

The equipment does not provide for the use of consumables in the sense of parts replaceable by the target customer.

Contact your distributor or the manufacturer's customer service department for replacement parts. See the warranty and service section for contact information.

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13 Troubleshooting

If there are problems with the child's fastening or adjusting the elements, the condition of the Velcro must be verified. They must not be dirty. There must be no parts between the fasteners that hinder the fastening process, e.g. foils or slippery upholstery.

In the event that it is not possible to fasten the rails on the back of the device, check that they have not been blocked by elements that should not be there.

Contact your distributor or the manufacturer's customer service department for replacement parts. See the warranty and service section for contact information.

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NOTE! When washing upholstery covers, pay special attention to velcro pads. In order to avoid upholstery damage, pay particular attention to detach the velcro during the washing process, and make sure that the velcro does not touch the surface of the upholstery. Do not wash the foam inserts!

14 Cleaning and maintenance

14.1 Recommendations concerning the cleaning and maintenance

Offered upholstery is made of a breathable fabric. Our products have been certified with the Oeko-Tex Standard 100 confirming the complete user safety, also when used by children. Fabrics used for our upholstery are free from any hazardous substances, such as pesticides, chlorophenols, formaldehyde, allergising dyes, prohibited azo dyes and extractable heavy metals. The Oeko-Tex Standard 100 is only granted to textiles, where all components have been tested and obtained positive results at each and every stage of the production process.

The upholstery coming into contact with the user has been covered with a special hydrophobic agent preventing the fluid penetration into the upholstery. The impregnate also facilitates maintaining the cleanliness of the upholstery. Slight dirt can be removed with a damp cloth. In case of more significant contaminations, you can simply wash the upholstery.

- It is crucial to remove the foam inserts from covers before washing.
- Covers should be hand washed or washed in a washing machine (with a drum) at 40 °C temperature.
- Use washing agents with the PZH /National institute of Hygiene/ certificate anticipated for delicate products in the amount stated on the package.
- Use a curd soap or special chemical agents in case of children prone to allergies.
- Dewatering - do not squeeze, it is possible to do a short spin dry programme.
- Drying - hang to dry at room temperature.

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The foam insert should be:

- vacuum mechanically or with a soft brush,
- it is possible to clean the insert with a damp cloth with a gentle washing agent. Then leave the insert to dry thoroughly at room temperature.

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14.2 Disinfection

In case when the device is used by various users (e.g. in a rehabilitation centre), it is essential to use disinfecting agents. As far as manual disinfection of the device is concerned, it is recommended to use Incidin plus in 0.25% to 0.5% solution, or any similar disinfecting agent.

Please follow any usage instructions stated by the manufacturer.

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NOTE! It is crucial to cleanse the upholstery and handles before performing the disinfection.

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15 Disposal of product

If the user abandons further use of the product, he is obliged to dispose of the product in accordance with environmental regulations. Disinfection must be carried out before disposal, as a product which has not been disinfected in accordance with current environmental regulations is hazardous waste.

The user may:

- contract a company that has the required authorizations to receive the equipment for decommissioning.
- in the event of disposal of the product, the plastic and metal parts used must be disposed of separately in accordance with environmental protection requirements and waste separation requirements.
- If you have any questions, please contact your local authorities, waste disposal companies or the appliance manufacturer.

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16 Conditions of use, storage and transport

The product is intended for use as portable sitting.

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For the necessary dimensions and weights of the unit for handling and transport, please refer to the technical data section.

Transport, handling and storage are best carried out on an empty folded product so that no damage is caused to the product or third persons.

TravelSit is supplied in backpack and when it is not use it should be stored and transported in this backpack.

How to assemble the unit and how to remove components, if necessary, is discussed in the Use and adjustment chapter.

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CAUTION! It is not permitted to carry the device while patient use the device.

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CAUTION! During use, the product may change the temperature of the patient/user interface depending on the external heat sources to which it has been exposed (e.g. sunlight)

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CAUTION!

The device can be stored/transported and used at temperatures between -10°C and +30°C and relative humidity between 10% and 60%; however, it is recommended that the device is stored/transported at room temperature and humidity.

If the device has been stored/transported in high ambient temperatures and has been exposed to direct sunlight, ensure that the device is at a safe temperature for use, i.e. the carer should check that the temperature of the device is not too high before the user has any contact with the device.

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17 Warranty, Service

In case when any defects or damages should occur, cease to use the device immediately and contact the seller or the manufacturer. Secure the damaged device in order to prevent further extension of the damage. Do not undertake any attempts to individually repair the device. do not replace any original parts with spare parts

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Dane kontaktowe serwisu:

developed individually or obtained from any source other than the one recommended by the manufacturer.

- The manufacturer determines the product life to be 5 years.
- The post -warranty service of the device is performed by the manufacturer.

Contact details of the service department:

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- Current address details can be obtained on the following website: www.liwcare.pl.
- Terms of the warranty have been specified in the warranty card.